CAPD Exchange Procedure

stay•safe®

Important:

With this and all Peritoneal Dialysis procedures, always maintain aseptic technique. This information is provided for quick reference only. This poster is not intended to replace the advice or training from your physician or PD nurse. Please refer to the Instructions For Use (IFU) for detailed information on device description, instructions, contraindication warnings and precautions.

Clean the work surface and gather the supplies needed for the exchange:

- Mask
- Warmed peritoneal dialysis solution
- stay•safe® cap
- stay•safe® organizer

Retrieve catheter and close your extension set clamp. Put on your mask. Everyone in the room must also put on a mask. Wash your hands. Open the solution bag by tearing from a notched edge of the package overwrap. Wipe off any moisture on the bag surface.

Check your solution:

- Strength: Make sure that you have the correct dextrose concentration
- Clarity: Check that the solution is clear
- Amount: 2, 3 or 5L volume
- Leaks: Squeeze the bag to check for leaks
- Expiration: Check the expiration date

Pick up the stay•safe® disc with its colored plastic cover. Turn the blue dial on the disc counterclockwise until it fits into the cut-out portion of the plastic cover. Remove the plastic cover after the dial is in the cut-out. The blue dial will be at Position 1 (●●●●).

Position the organizer at the edge of your work surface or on the IV pole. Place the disc into the organizer as shown.

Hang the solution bag on an IV pole and place the drain bag on the floor. Break the frangible in the solution bag outlet port.

Remove the stay•safe® cap from its package. Place the cap in one of the two notches on the organizer.

Wash hands again or apply antibacterial hand gel, per your clinic protocol. Place the end of your extension set in the other organizer notch. Remove the protective cap from the stay•safe® disc. Discard the cap.

Unscrew your extension set from its cap. The cap will remain in the organizer.

Immediately connect the extension set to the stay•safe® disc. After making this connection, you may remove your mask.

Open the clamp on your extension set. You will start to drain as soon as the clamp is opened.

When your drain is complete, turn the dial to Position 2 (●●●●). This will start the flush from the fill bag to the drain bag. Make sure the line between the solution bag and disc fills completely.

After about 5 seconds, turn the indicator to Position 3 (●●●●). This will start the fill to your abdomen.

When the fill is complete, turn the indicator to the very end of Position 4 (●●●●●). This will insert the closure pin into the end of the extension set. You will feel a "click" when you get to the last position.

Close the clamp on your extension set. Unscrew the protective cover from the new stay•safe® cap. **Note:** If you wish to stop the procedure during any step, please turn the blue dial to "●●●●●" to activate the safety lock with the PIN.


Unscrew your extension set from the disc.

Immediately attach your extension set to the new stay•safe® cap. Remove your capped extension set from the organizer and secure it to your abdomen.

If desired, place the protective cover from your new stay•safe® cap on the used cap. Then remove the cap from the organizer and place the other end of the cover on the disc to prevent drips. Record your exchange on your flow sheet and discard the used tubing and solution as instructed by your facility.

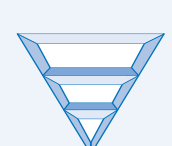Fresenius Medical Care

Fresenius Medical Care North America · 920 Winter Street · Waltham, MA 02451 · 800-662-1237 · www.fmcna.com
Fresenius Medical Care, triangle logo and stay•safe are trademarks of Fresenius Medical Care Holdings, Inc., or its affiliated companies
© 2005, 2009 Fresenius Medical Care North America. All rights reserved.

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