

## Liberty® Cycler Patient Skill Checklist

Patient Name: \_\_\_\_\_ Date: \_\_\_\_\_

Clinic: \_\_\_\_\_ RN Name: \_\_\_\_\_

Skill Demonstrated		√ indicates skill performed correctly		
		Simulated	1st Tx	2nd Tx
<b>Setting up the Liberty Cycler for Treatment</b>				
1	Finds and turns on the power switch to the Liberty.			
2	When the Liberty shows the "Ready" screen, checks the treatment settings.			
3	If treatment settings are correct, presses "OK" to start treatment.			
4	Follows the steps on the screens exactly. Presses "Next" when indicated on each screen.			
5	Gathers correct supplies for the treatment per clinic instructions.			
6	Puts on mask and ensure that everyone else in the room puts on a mask.			
7	Washes hands thoroughly per clinic procedure.			
8	Opens the cassette package with the tubing and removes it from the package. Places the blue patient line in the stay•safe® organizer.			
9	Locates and opens cassette door.			
10	Correctly snaps the cassette into place. Does not touch the door sensor when loading the cassette. Closes the door.			
11	When the cycler instructs, connects the lines to the bags.			
12	Places a solution bag on the heater tray in correct position. Follows screen instructions.			
13	When the cycler instructs, breaks the cones. Clamps any unused lines.			
14	Verifies the bags are connected per screen indicator.			
15	After flush is complete, opens the blue line clamp.			



## Liberty® Cyclor Patient Skill Checklist

Skill Demonstrated		√ indicates skill performed correctly		
		Simulated	1st Tx	2nd Tx
16	Checks the blue patient line to make sure it is primed. Primes a second time if Integrated Set is used.			
17	Closes the blue line clamp.			
18	Using the stay•safe organizer, connects the blue line to patient. Opens the blue line clamp and catheter clamp.			
19	Presses "Next" to start treatment.			
20	During drain 0, holds the drain line with chamber upside down to fill it with effluent. Checks the effluent for clarity.			
<b>Discontinuing the Liberty Cyclor Treatment</b>				
1	Follows the steps on the screens exactly. Presses "Next" when indicated.			
2	At the end of the treatment, closes the cyclor blue patient line clamp.			
3	Turns the blue patient connector clockwise to unlock, then pushes the trigger to set the pin. Clamps the blue patient extension set line.			
4	Washes hands or cleans with a waterless antiseptic.			
5	Checks to assure a new cap is in the left side of the stay•safe® organizer.			
6	Places the blue patient line connected to the catheter into the center of the stay•safe organizer.			
7	Removes the protective cover from the new cap.			
8	Unscrews the extension set from the bluepatient line in the organizer.			
9	Immediately connects the extension set to the new cap.			



## Liberty® Cycler Patient Skill Checklist

Skill Demonstrated		√ indicates skill performed correctly		
		Simulated	1st Tx	2nd Tx
10	Removes the capped extension set from the organizer.			
11	Presses the "Next" button on the cyclor.			
12	Closes all clamps on the cyclor lines. Removes the cassette from the cyclor and closes the door.			
13	Presses "Next" to continue.			
14	Reviews and explains the treatment summary to the nurse.			
15	Presses "Next" to continue.			
16	Turns off the cyclor.			
17	Cleans the cyclor with (1:100) bleach solution and rinses it off.			
<b>General Machine Knowledge</b>				
1	Identifies heater/scale, control panel, door, and on/off switch.			
2	Demonstrates how to handle alarms by pressing stop to mute, identifying alarm code, and troubleshooting as taught.			
3	Can name the four hard key buttons.			
4	States the colors of the status bar and what each color indicates.			
5	Demonstrates how to check "Treatment Settings".			
6	Demonstrates how to check "My Records".			
7	Demonstrates how to set "Options" for cyclor operation, including SI units.			
8	Identifies the color of the cassette line clamps and what each color indicates.			



## Liberty® Cycler Patient Skill Checklist

Skill Demonstrated		√ indicates skill performed correctly		
		Simulated	1st Tx	2nd Tx
9	If able, lifts the cyclor properly when moving it.			
10	Demonstrates how to accept or reject the settings from the IQdrive.			
11	Demonstrates how to change treatment parameters on the "My Settings" screen.			
12	Presses "OK" to save changed settings prior to leaving a screen when a change has been made during programming.			
13	Differentiates between connected and unconnected lines in the "verify bags" screen.			
14	Troubleshoots alarms as instructed.			
15	Identifies location of Technical Support 800#, Software Version and Serial Number on Liberty cyclor.			
16	Writes down the alarm code prior to calling Technical Support.			
17	Describes the function of the "details" button.			
18	Demonstrates how to insert the IQdrive into the cyclor.			
19	Progresses appropriately after a power failure.			

Date(s) skills demonstrated: \_\_\_\_\_

Patient Signature: \_\_\_\_\_

RN Signature: \_\_\_\_\_

Please refer to the Liberty Cyclor User's Guide for detailed information on device description, instructions, contraindications, warnings and precautions.



**Fresenius Medical Care**

Fresenius Medical Care North America  
 920 Winter Street  
 Waltham MA 02451  
 800-662-1237  
 www.fmcna.com

© 2009 Fresenius Medical Care North America. All rights reserved  
 P/N 101093-01 Rev. 00 08/2009

success@home™

